




Welcome to your Assurelink Smart Home Care System. Home, with its comfort and familiarity, plays a central role in all our lives. That's why it's natural to want to stay in it for longer. When it comes to people who love and care for each other, it's also natural to want to stay in the know, assuring and linking us to each other. Our system provides reassurance and support for home living independence in an easy to setup, out of the box solution.

For further information please visit our website [www.assurelink.co.uk](http://www.assurelink.co.uk) where you can find a copy of this guide, a Q&A section and contact details.

[www.assurelink.co.uk](http://www.assurelink.co.uk)  
[info@assurelink.co.uk](mailto:info@assurelink.co.uk)  
01278 874677

## WHAT'S IN THE BOX

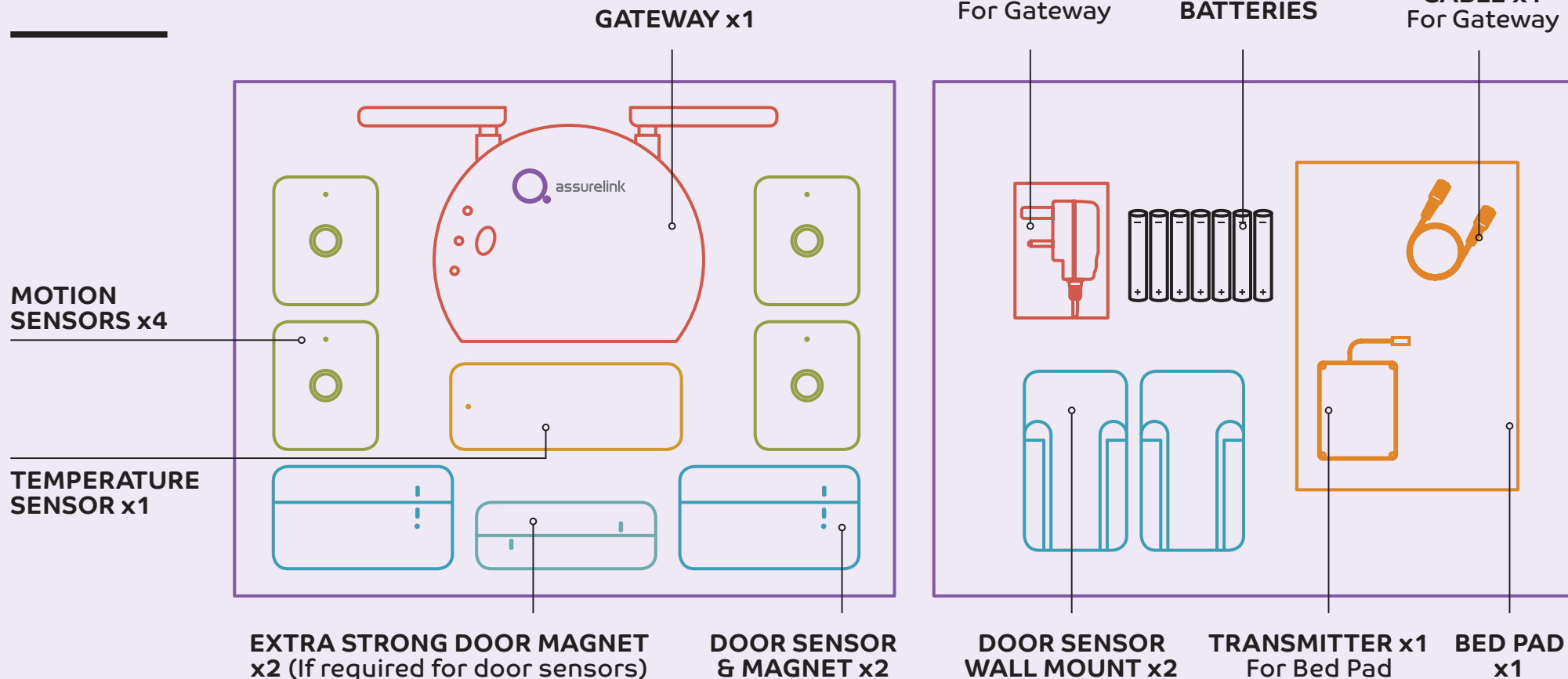

## WHAT GOES WHERE

### WE ADVISE THAT:

- The motion sensors are setup in the bathroom, kitchen, living room and bedroom according to the stickers on them.
- The temperature sensor is placed in a central area such as the hallway.
- The door sensors are setup on the front and back doors.
- The bed pad under the sheet on the bed.
- The gateway near the WiFi router.


### PETS

We understand that pets can be an important part of some households. Their movement will also be detected, as with other motion sensors, leading to false readings. To help minimise this please see the diagram showing the placing of the sensors.

### SENSOR REACH

Place your sensor above a shelf or furniture to avoid the detection of ground level pets.


## HOW TO RESET YOUR GATEWAY

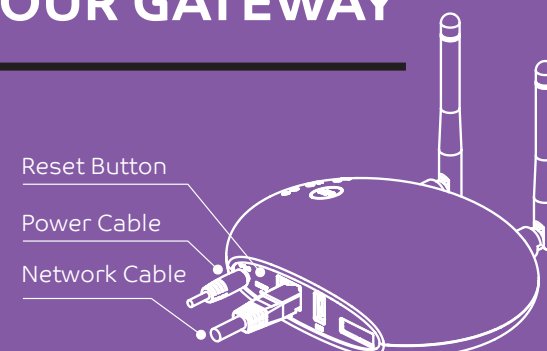

- If your app account does not connect, ensure the internet and power cables are connected to gateway
- Use the end of a paperclip to press the reset button
- Hold for 5 seconds
- Reconnect internet and power cables after indication lights have stopped flashing
- Now, you can retry the normal setup procedures with an existing or new app account.


# HOW TO INSTALL

## 01 GATEWAY

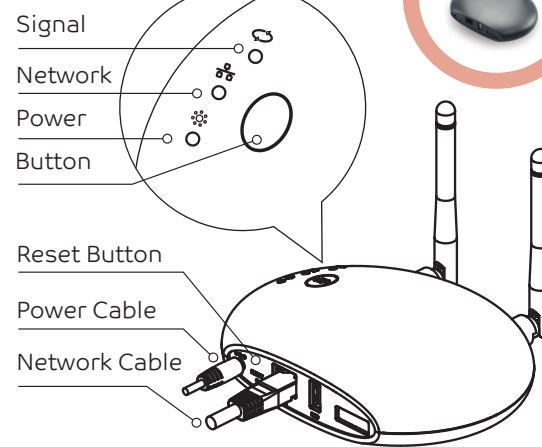

- Connect network cable
- Connect power cable


## 02 BED PAD

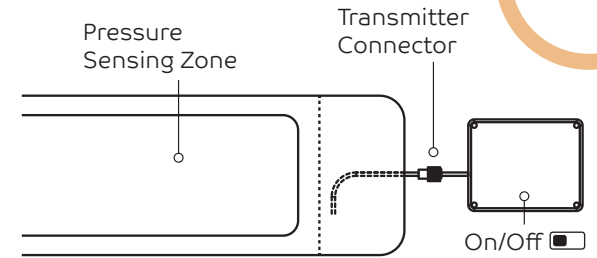

Cover bed pad with a bed sheet


- Plug transmitter into bed pad
- Switch on transmitter
- Cover transmitter inside side pocket
- Press bed pad for 15 seconds to activate
- Place bed pad at shoulder or hip level

## 03 TEMPERATURE SENSOR

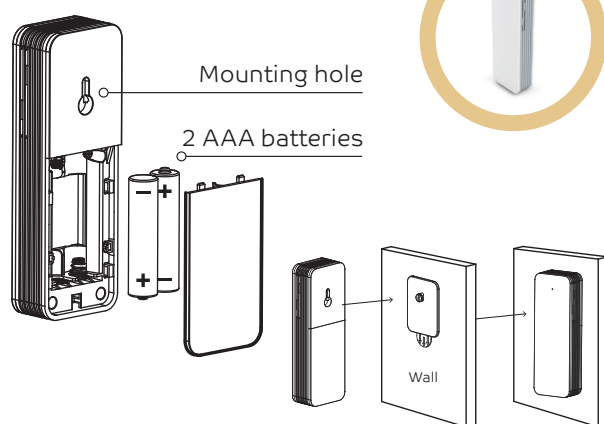


- Insert 2 x AAA batteries
- Install with wall hook and double-sided tape
- Install in a well-ventilated indoor location
- To remove, press and pull double-sided tape out

## 04 MOTION SENSOR

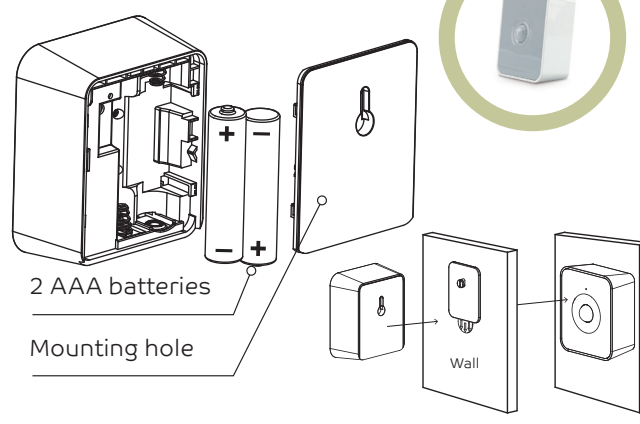

- Insert 2 x AAA batteries
- Install with wall hook and double-sided tape
- Install in living room, kitchen, bathroom and bedroom
- To remove, press and pull double-sided tape out


## 05 DOOR SENSOR

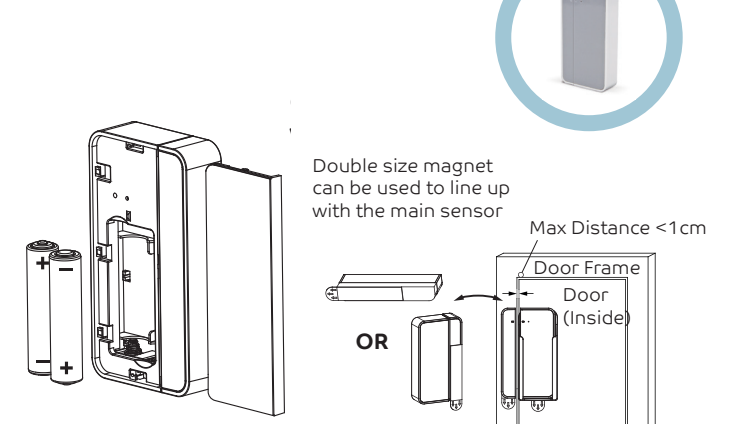

- Insert 2 x AAA batteries
- Install main sensor into wall-mounted case
- Install main sensor at the door's opening side
- Install magnet in the door frame – match the markings on both parts

## 06 DOWNLOAD ASSURELINK SMART HOME CARE SYSTEM APP

You can start using Assurelink Smart Home Care System by just completing a few steps on your mobile app.

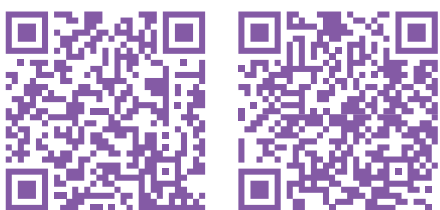

Available on the **App Store** **Google play**



## INSTALLATION GUIDE

## SMART HOME CARE SYSTEM

